
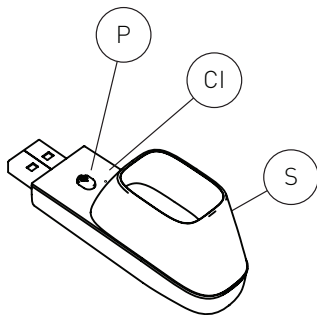
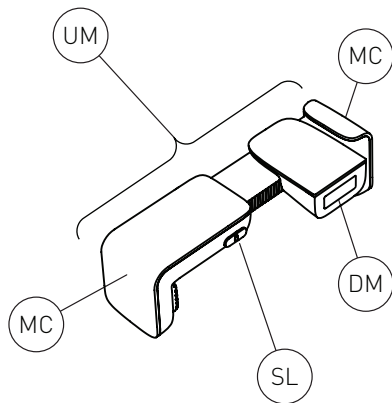
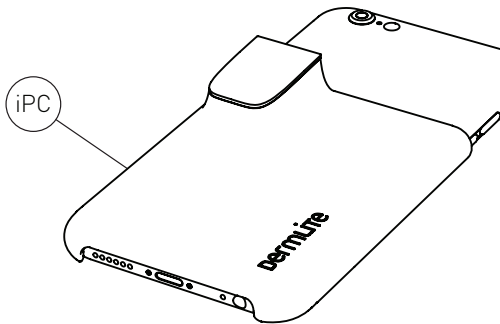
# dermlite®

DIRECTIONS FOR USE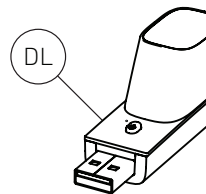

# Instructions

**CAUTION:** Prior to using this product, read these instructions and keep them in a safe place for future reference.

**CAUTION:** This product is designed for external examination only.

This product is a smartphone-compatible magnifier designed to allow skin lesion photography with high magnification and clarity. It may be attached to a smartphone using one of the included adapters, positioned over the smartphone's camera, and placed on a skin area of concern.

## Directions for Use

**CAUTION:** Do not look directly into the bright LED light. Patients must close their eyes during facial use.

**IMPORTANT:** Before initial use, please charge the device for a minimum of 1 hour. See "Battery & Charging" for details.

This product includes a DermLite lens (DL) and two different types of adapters compatible with various smartphones. If the included iPhone Case (iPC) will not slide onto your iPhone, as shown in Fig 1., choose the Universal Mount (UM), which is designed to fit on virtually any smartphone 2.3" - 3.1" (58-78mm) in width. For best results, you may choose to remove any case you have on your smartphone.

## Universal Mount

To attach the Universal Mount (UM) to your smartphone, place it over the back of your phone and firmly squeeze the Mount Clamps (MC) together. Move the Slide Lock (SL) into the locked position to hold the Mount Clamps (MC) in place. Plug the DermLite Lens (DL) into the DermLite Mount (DM). Slide the DermLite Mount (DM) left or right such that the DermLite Lens (DL) is horizontally centered over your phone's camera. To center the DermLite Lens (DL) vertically over the phone's camera, move the Slide Lock (SL) into the unlocked position, and loosen the Mount Clamps (MC). Adjust the vertical position such that DermLite Lens (DL) is vertically centered over your phone's camera. Squeeze the Mounts Clamps (MC) firmly together and move the Slide Lock (SL) into the locked position to hold the Mount Clamp (MC) securely in place.

## Taking a Picture

Once you have chosen a spot to photograph, lightly moisten it and its surrounding skin with hand sanitizer or similar liquid. This increases the translucency of the skin and improves the quality of the image. With this product attached to your smartphone, place the black spacer (S) directly on the skin and push the power button (P) to turn on the DermLite Lens (DL). Capture the image with your smartphone. Should your smartphone focus on the hair instead of the skin surface, reapply moisture to flatten the hair or shave the area.

## Battery & Charging

**CAUTION:** This product uses a special lithium battery, which can only be purchased from 3Gen or from an authorized 3Gen dealer. Do not under any circumstances use another battery other than the one designed for this unit. Doing so could result in damage to the unit.

To conserve energy, this product is designed to automatically turn off after 1 minute, and the battery can be expected to supply power for about 30 one-minute sessions. If it shuts off prior

to 1 minute, it may need to be recharged. To charge, plug the DermLite Lens (DL) into any USB port. While charging, the Charge Indicator (CI) will glow red; when fully charged, it will appear green.

## Troubleshooting

Please check our website ([www.dermlite.com](http://www.dermlite.com)) for the most current troubleshooting information. If your product requires servicing, please go to the "Service & Support" pages at [www.dermlite.com](http://www.dermlite.com), where you can complete a "Repair Request" form. Or, contact your local 3Gen dealer.

### No Light

Fully recharge the battery and check again. If the condition persists, your product may require servicing.

### LED Fails

The LEDs used in your product are designed to last over 100,000 hours. If any one of the LEDs fails, your product requires servicing.

### Mechanical Malfunction

Please contact 3Gen to arrange for service.

## Care and Maintenance

Your product is designed for trouble-free operation. There are no adjustments to be made nor is any special servicing required. Never attempt to open the device for any reason other than battery replacement.

### Cleaning and Sterilization

The exterior of your product may be cleaned or disinfected by wiping it with isopropyl alcohol (70% vol.). Do not use alcohol or disinfectants in the optical areas of the unit. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

### Cleaning

The DermLite lens should be treated as high quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals.

### Components include:

DermLite Lens unit, iPhone adapter, Universal Mount.

### Warranty

2 years for parts and labor.

Serial number is engraved into the USB plug.

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Dermlite and 3Gen are registered trademarks of
©2016 by 3Gen, Inc. | All rights reserved. | REF A DLM-1701
3Gen, Inc. | 31521 Rancho Viejo Road | Suite 104
San Juan Capistrano, CA 92675 | U.S.A.