



MAINTENANCE INSTRUCTIONS

Post this in your sterilization room

DENTAL LOW SPEED E-TYPE ATTACHMENT

1 SCRUB

Remove the attachment from the motor and remove bur.
Close the chuck on straight attachments.
Scrub the attachment with a fine wire brush or a soft brush under running water to remove "bio burden".
Wipe and dry with paper towel.



2 SPRAY

Using the correct lubrication nozzle, spray MK-dent Premium Spray Lubricant and Cleaner into the back end after every use.
Spray for approximately 2 seconds.



3 EXPEL

Connect the attachment (with head, without bur) to the motor and loosely cover front end with paper towel.
Run the attachment for 10-15 seconds with the head down to expel debris and excess lubricant. Wipe off excess lubricant with alcohol.

Repeat process until fluid comes out clear.

Note: Use pure rubbing alcohol only when wiping off excess lubricant.

Many over the counter alcohol wipes have other solutions added to them.

This can cause spotting of the exterior and severe damage to the internal parts of the handpiece.



4 REMOVE

Remove head from the attachment. Wipe off excess lubricant.
For straight nosecones, make sure the chuck collars are in the closed position before sterilizing.

Note: For automatic purging stations, you can skip step #2 and #3.

Please follow manufacturer's recommended maintenance instructions.



5 STERILIZE

Do not re-assemble head to attachment prior to sterilization.
Insert attachment into pouch, bag or wrap.
Follow sterilizer manufacturer's instructions for autoclave/chemiclave.
(Most products are sterilizable at temperature of 135°C/275°F.)

Note: Never remove components from sterilizer until drying cycle has ended.




6 ALLOW TO COOL

Remove the attachment from the sterilizer and let them cool to room temperature.

Note: Never cool handpieces under running water or by covering/wrapping with a wet towel




LUBRICATION TOOLS

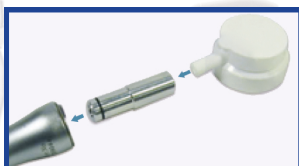

LT 1013
E-type Connection
Lubrication Tool
\$8.90



LU 1011
MK-dent Premium
Handpiece Synthetic
Spray Lubricant
\$32.00

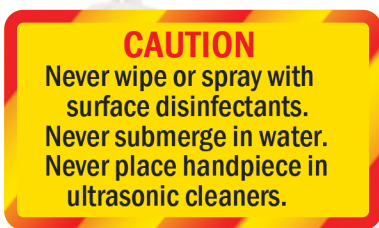


HANDPIECE SOLUTIONS
6785 S. Eastern Ave., #11
Las Vegas, NV 89119
Toll Free: 1 (888) 488-3885
www.HandpieceSolutions.com

rev: 09/21