



MAINTENANCE INSTRUCTIONS

Post this in your sterilization room

DENTAL LOW SPEED HEAD ATTACHMENT

1 SCRUB

Remove the contra angle attachment from the motor and remove bur.
Scrub the attachment with a fine wire brush or a soft brush under running water to remove "bio burden".
Wipe and dry with paper towel.

2 SPRAY

Remove the head from the attachment.
Using the correct lubrication nozzle, loosely cover the head attachment and spray MK-dent Premium Spray Lubricant and Cleaner into the head for approximately 2 seconds..

Note: Contra angle with the head attached can be lubricated together.

MIDWEST TYPE

Remove the Midwest type head from the attachment.
Separate the head and the intermediate piece.
Wipe intermediate piece with alcohol.
Using the LT 1014 nozzle, loosely cover the head and spray lubricant into the back end of the head for about 2 seconds.



3 EXPEL

Assemble the intermediate piece (for Midwest type) to the head, then connect head assembly to the attachment and motor.

Loosely cover the front end with paper towel.

Run the attachment for 10-15 seconds with the head down to expel debris and excess lubricant. Wipe off excess lubricant with alcohol.

Repeat process until fluid comes out clear. Remove head from attachment and wipe off excess lubricant.

Note: Use pure rubbing alcohol only when wiping off excess lubricant.

Many over the counter alcohol wipes have other solutions added to them that can cause spotting of the exterior and severe damage to the internal parts of the handpiece.

Note: For automatic purging stations, you can skip step #2 and #3. Please follow manufacturer's recommended maintenance instructions.

4 STERILIZE

Do not re-assemble head to attachment prior to sterilization. Insert attachment into pouch, bag or wrap.

Follow sterilizer manufacturer's instructions for autoclave/chemiclave.

(Most products are sterilizable at temperature of 135°C/275°F.)

Note: Never remove components from sterilizer until drying cycle has ended.

5 ALLOW TO COOL

Remove the head from the sterilizer and let it cool to room temperature.

Note: Never cool handpieces under running water or by covering/wrapping with a wet cloth.



CAUTION

Never wipe or spray with surface disinfectants.
Never submerge in water.
Never place handpiece in ultrasonic cleaners.

LUBRICATION ADAPTERS



LT 1013
E-type Connection
Lubrication Tool
\$8.90



LT 1014
For cleaning chucks
and general
purpose tool
\$8.90



LU 1011
MK-dent Premium
Handpiece Synthetic
Spray Lubricant
\$32.00



6785 S. Eastern Ave., #11
Las Vegas, NV 89119
Toll Free: 1 (888) 488-3885
www.HandpieceSolutions.com

rev: 09/21